PORTUGUÊS

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Ref. 10944-F. 02/2019

ENGLISH

CALLUSE PROTECTIONS with EPITHELIUM™ Activ

INDICATIONS

To protect the skin and provide pain relief from hard skin and calluses on the front of the foot etc.

CONTRAINDICATIONS

- Do not use on a skin lesion.
- In the event of an allergic reaction, or substantial irritation, the treatment should be stopped.

PRECAUTIONS FOR USE

- Wash before first use.
- Protections should be applied to clean, dry skin.
- Remove a protection from the metallic envelope.
- Remove the protective film covering the adhesive side.
- Apply the adhesive surface to the skin, making sure that the affected area is well covered.
- Apply light pressure to the protection to ensure that it has adhered to the foot.
- Gently cover with socks, stockings or tights.
- Keep the protection in place during the day.
- Remove at night, place on the tray provided for this purpose and return to the metallic envelope until the next day.

For optimum results it is recommended that the protection is applied everyday until the hard skin has disappeared.

WHAT IT IS USED FOR

- The effectiveness of these self-adhesive protections is due to EPITHELIUM™ Activ gel which evenly distributes pressure: walking becomes more comfortable as soon as they are applied.
 - Very effective, and yet very thin, the protections may be worn in all types of shoes (formal, sports, house etc.).
- These protections have been tested and approved for use by diabetics.

CLEANING

Protections may be washed with soap and water. Do not rub, allow to air-dry.

COMPOSITION: 84% silicone, 16% polyamide/elastane fabric.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. MILLET Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2006.

	Hand washable.		Do not iron or steam.		Do not dry clean.
	Do not bleach.		Do not tumble-dry.		See packaging.

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