ÓRTESE POLEGAR FLEXÍVEL DIURNA

RIZARTROSE com EPITHELIUMFLEX®

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MILLET Innovation ZA Champgrand
26270 Lorient/Drôme - FRANCE

contact@epitact.com
www.epitact.com

FLEXIBLE THUMB BRACE FOR RHIZARTHROSIS with EPITHELIUMFLEX®

INDICATIONS

Helps to relieve articular arthrosis pain at the root of the thumb by restricting microinstability.

CONTRAINDICATIONS

- Do not use on a skin lesion.
- In the event of severe oedema: look closely at the splint during use. Remove the splint if it is too tight and impedes circulation. When in doubt seek advice from your GP or your pharmacist.

PRECAUTIONS FOR USE

- Wash before first use.
- This product is a day use splint.
- Two versions are available: right hand and left hand splints (stitching is inside the hand splint).
- Do not use near a flame or incandescent body.

WHAT IT IS USED FOR

The EPITACT® flexible proprioceptive splint (orthosis) enables maintaining full function of the hand throughout the day. It is used as a complement to rigid splints which are more suitable for nocturnal use.

WEARING THE SPLINT

Attach the splint to your hand by sliding your thumb into the thumb section. Stitching must be inside the splint. For optimal comfort, adjust the position of the splint by slightly moving the stitching one way or another in the centre of the splint.

CLEANING

- Washing after use at 30°C in the washing bag supplied or wash by hand (without rubbing).
- Do not use stain removers (acetone, solvents, etc.)
- Discard and replace the device as soon as fabric or seam damages occur.
- Leave to dry naturally. Do not iron.

COMPOSITION: 86% polyamide/elastane fabric, 14% silicone.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. MILLET Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2013.

	Machine washable at 30°C.		Do not iron or steam.		Do not dry clean.
	Do not bleach.		Do not tumble-dry.		See packaging.

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